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Instruction for use

Please read this information carefully because it will help you to use HYLO NIGHT® properly.

HYLO NIGHT®

For improvement of the tear film and protection of the eye's surface in case of dry eye disease

Eye ointment without preservative

PRODUCT DESCRIPTION

HYLO NIGHT® is a sterile eye ointment with Vitamin A for ocular use.

COMPOSITION

1 g HYLO NIGHT® contains 250 I.U. retinol palmitate (vitamin A) and liquid paraffin, light liquid paraffin, wool fat, white soft paraffin.

1 dosage = 1 strip of eye ointment (approx. 1 cm long) corresponds to approx. 5 I.E. vitamin A.

One tube contains 5 g of eye ointment.

INTENDED PURPOSE

HYLO NIGHT® improves the tear film and protects the eye's surface in case of dry eye disease.

MEDICAL INDICATION

Dry eye disease

SUITABLE FOR

HYLO NIGHT® is suitable for patients of all ages without any limitation of age. Based on existing knowledge, children, pregnant and breast-feeding women can also use HYLO NIGHT®.

BENEFITS, PERFORMANCE AND EFFECTIVENESS

The clinical benefit is the relief from symptoms of dry eye disease. HYLO NIGHT® is a particularly soft and smooth eye ointment that spreads very well on the eye's surface. The vitamin A is a natural component of the tear film supporting the eye ointment's good tolerance and helps the ointment to mix well with the available tears. Thus, the sensation of burning, dry or tired eyes is relieved. The eye lids slide easily over the surface of cornea and conjunctiva.

Air conditioning, e.g. in cars or airplanes, wind, cold, strong solar irradiation, bad air or cigarette smoke often cause dry or itching eyes with a feeling of dryness or foreign bodies.

Also concentrated watching, e.g. when working long time in front of the screen, watching TV or when driving at night can cause these symptoms. If you are wearing contact lenses, there is an even higher risk for these impairments of the eye's condition.

HYLO NIGHT® relieves these symptoms and constitutes a convenient long-lasting film on the eyes. The fatty consistency of the eye ointment causes a slight impairment of vision. Therefore, HYLO NIGHT® should be applied before going to bed. Possible discomfort when opening the eyes for the first time in the morning can be prevented by using HYLO NIGHT®.

GENERAL INSTRUCTIONS FOR USE

Eye ointments should be used avoiding a contact of the tube-tip with the eye or the skin.

Only use HYLO NIGHT® if the sealing of the folding carton was intact before the first use (🔒).

Wash your hands and stay in front of a mirror.

Unscrew the cap covering the nozzle. Please apply the eye ointment by gently squeezing the tube without folding and reeling.

Tilt your head backwards and gently pull on the lower eyelid using a finger of one hand.

With the other hand hold the tube in an upright position above the eye. Instil one strip of eye ointment into the gap formed between the eye and eyelid by gently squeezing on the tube.

Close your eye gently then blink normally.

Replace the cap firmly back onto the tube.

HYLO NIGHT® can be used in combination with lubricating eye drops. However, as it should not be used with other eye drops at the same time, in this case, HYLO NIGHT® should be used about 30 minutes after the application of the eye drops. In case you need to use a further eye ointment please keep an interval of at least one hour between both applications.

HYLO NIGHT® can be used for 6 months after first opening and should not be used beyond expiration data (📅). Please contact the manufacturer, your pharmacist or ophthalmologist in case of any questions about correct use of this medical device.

DOSAGE

The frequency of use depends on your personal complaints and habits. HYLO NIGHT® is used once a day before going to bed. In more severe dry eye sensation HYLO NIGHT® is usually applied three times a day into each eye.

If you have the feeling to use HYLO NIGHT® more often ask your doctor.

The duration of treatment is not limited.

If you follow the dosage instructions an overdosing with HYLO NIGHT® is practically not possible as the target tissue is the ocular surface resp. the conjunctival sac which has only a limited volume.

PRECAUTIONS AND WARNINGS

If you do not feel better or if you feel worse after several days of use talk to your doctor or pharmacist.

In case you use HYLO NIGHT® during the day, your vision will be impaired for some time because of the fatty consistency of the eye ointment. Until this clears up, you should not drive a car, use machines or work without stable footing.

Only the same person should be treated with one tube HYLO NIGHT® (👤).

SIDE EFFECTS

The ingredient wool fat may cause localized skin reactions (e.g. contact dermatitis). Please check before use if you have a known hypersensitivity to wool fat.

In very rare cases hypersensitivity reactions like red, burning, itching eyes, excessive tearing or other temporary local irritations were reported which improved after the treatment with HYLO NIGHT® was stopped.

In case a serious incident has occurred during the use of this medical device or as a result of its use, please report it to the manufacturer (🏭) and to your national authority.

CONTRAINDICATIONS

HYLO NIGHT® should not be

- applied if you are hypersensitive against one of the ingredients

- used while wearing contact lenses.
- used directly after ophthalmological interventions or in acute injuries of the eye as no clinical studies confirm this indication.

STORAGE

Do not store above 25 °C (🌡️^{25 °C}).

Keep HYLO NIGHT® out of the reach and sight of children.

Any medical device you no longer use should be thrown away via household waste. This measure will help to protect the environment.

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Your opinion about our product is relevant for us! We are looking forward to get your feedback via mail or via www.ursapharm.de.

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🏭 URSAPHARM Arzneimittel GmbH